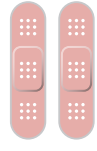
Collection 1

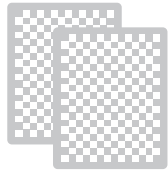
Includes:

2 Alcohol wipes

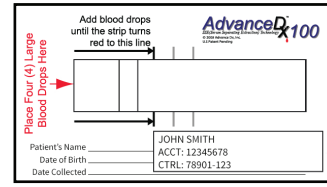
2 Band-aids

2 Gauze

2 Lancets

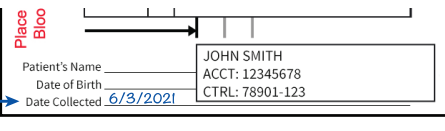
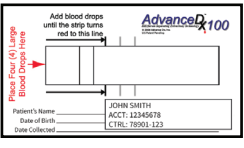
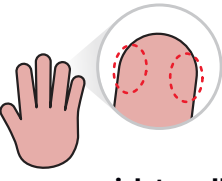
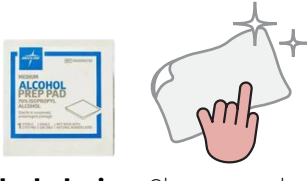
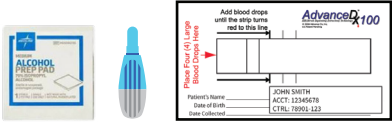
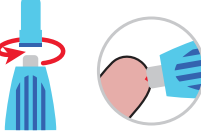
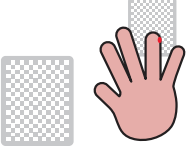
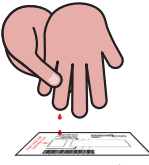
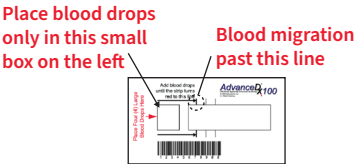
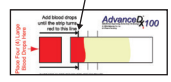
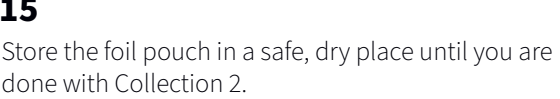
1 Sample collection card

1 Foil pouch with moisture absorbers

Collection 1 can be collected at any time of the day and does not require fasting. Please be prepared to ship **Collection 1** and **Collection 2** to the laboratory for testing within 24 hours of collection.

- 1**  **Read through the instructions completely before collecting the sample.** Test results may be invalid if sample is not collected properly.
- 2**  **Open the bag marked “Collection 1” and place all the contents on a clean, dry surface.** Locate the sample collection card, **confirm your name is properly printed on the label and write today’s date in ink on the card.**
- 3**  **Wash your hands with soap and warm water.** Rub your hands together for 10 to 15 seconds to help increase blood flow. Dry your hands thoroughly.
- 4**  **Place the sample collection card in front of you.** Do not touch the middle area of the card with your hands. Locate the gauze, one alcohol wipe, one blue and white lancet and a bandaid and set all near you.
- 5**  **Choose the finger you wish to collect from.** The best site for a finger prick is on the side of the fingertip of the third (middle) or fourth (ring) finger of your non-dominant hand. Your index finger and pinky will work fine too.
- 6**  **Open one alcohol wipe.** Clean your chosen finger with the alcohol wipe by rubbing the wipe over the entire fingertip. Let the finger dry for 15 seconds.
- 7**  **On the same side as the hand that you cleaned with the alcohol wipe, rest your elbow on a solid surface such as a countertop or table and hold your palm up.**
- 8**  **Twist and pull the blue cap off of the lancet.** Hold the lancet between your thumb and forefinger. Position the lancet tip against the rounded side edge of your finger and then press down firmly until you hear a “click.” Then release the lancet and set aside. Blood should begin to form on your finger. **Note: Each lancet can only be used once.**
- 9**  **Take a clean gauze pad and wipe away the first drop of blood.** Face your palm down and place the specimen collection card underneath your hand.
- 10**  Hold your finger over the left square of the collection card. Gently apply pressure across your palm and along the finger up to the puncture site.
DO NOT drop blood on the rectangle on the right side of the card directly. The blood from the left square will migrate to the right rectangle as intended.
DO NOT touch your finger to the blood collection card.
- 11**  **Without touching your finger to the collection card, add several drops of blood to the left side of the card until the blood spreads to the first line of the right rectangle on the collection card.**
- 12**  If more blood is needed, repeat steps 5-11 with another finger.
- 13**  Use the gauze pad to apply pressure to your finger to stop the bleeding. Place the bandaid on your finger. Allow your blood sample to air-dry on the blood collection card for one hour.
DO NOT place the card in direct sunlight while drying.
- 14**  After your blood sample has dried for one hour, place the blood collection card in the foil pouch and seal. **DO NOT** remove the moisture absorbers from the foil pouch.
- 15**  Store the foil pouch in a safe, dry place until you are done with Collection 2.
Note: Place all used collection materials, including lancet(s) and gauze, in garbage for disposal.

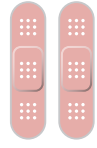
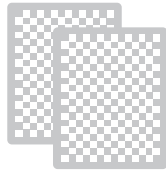
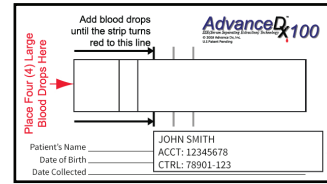
Now that you have completed **Collection 1**, locate **Collection 2** and follow the instructions provided in the collection bag. **Collection 2** can be collected at any time of the day and does not require fasting.

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