

Instructions for Use

Urine Monovette® With pH Stabilizer

Caution: The urine Monovette® contains a corrosive liquid that may cause severe burns to all areas of contact. Inhalation can cause irritation and burns, and it may be fatal if swallowed.

1. Friction fit the yellow suction tip securely onto the Luer end of the multi-adapter. (Figure 1)
2. Connect the multi-adapter to the urine Monovette® membrane cap via the three-pin locking mechanism: Guide the pins on the cap straight into the grooves of the adapter, then secure by twisting slightly clockwise. (Figure 1)
3. Place the suction tip down into the urine specimen and pull the urine Monovette® plunger to the black base fill line, drawing urine into the tube. The base fill line is located on the label near the bottom of the urine Monovette®. (Figure 2)
4. Hold the urine Monovette® upright, and fully extend the plunger to draw the remaining urine out of the yellow straw. (Figure 3)
5. Break off the plunger, and place it in the proper receptacle.
6. Remove the straw from the Luer adapter and place it in the proper receptacle.
7. Disconnect the multi-adapter from the urine Monovette® membrane cap by twisting slightly counterclockwise, and dispose of the adapter in the proper sharps container. (Figure 4)
8. Label and submit the specimen according to LabCorp's procedures.

Monovette® is a registered trademark of Sarstedt Inc.



Figure 1 — Connecting the Multi-adapter



Figure 2 — Drawing Urine Into the Tube

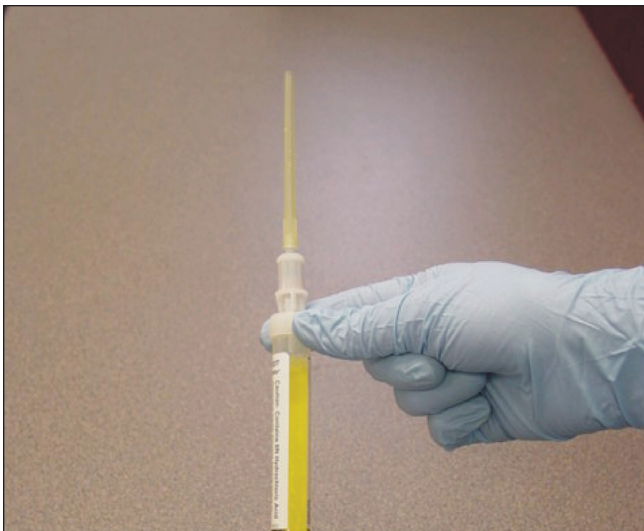


Figure 3 — Drawing the Remaining Urine out of the Straw



Figure 4 — Disconnecting the Multi-adapter